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ENGLISH

## ORTHOKERATOLOGY INSTRUCTION FOR USE

This notice contains important information, read it, and keep it. Orthokeratology rigid contact lenses are medical devices and should always be fitted by a contact lens specialist. For proper use, it is important that you follow all your specialist's instructions and usage tips on the packaging of your lenses and cleaning solutions.

- Your Orthokeratology rigid lenses are worn at night, during sleep, with eyes closed, and must be renewed every year. (Follow your specialist's recommendations).
- Orthokeratology treatment is done by corneal reshaping gradually throughout your night's sleep. However, it is recommended to sleep with your lenses for at least 6 hours for the treatment to be effective. Regardless of the wearer's age, night lenses require a regular sleep cycle and impeccable hygiene.
- To avoid altering the material or geometry of your lenses, it is essential to renew them according to the above modalities. Follow the appointments planned by your specialist that are essential to the control and follow-up of your adaptation.
- Wash your hands systematically before handling your lenses.
- Orthokeratology lenses are packaged in a unit case containing a Polyhexanide-based decontamination solution. It is equipped with an inviolability system until the first opening. If the packaging is damaged during the first use, the lens should not be used.

### Instructions

- Orthokeratology rigid lenses are lenses for orthokeratology treatment of myopia, astigmatism and hyperopia.
- The ophthalmologist or optometrist are the only professional people authorized to define the type and parameters of Orthokeratology lenses that should be prescribed based on your ametropia.

### Lens insertion

- Make sure your lens is clean, in good condition and always immersed in the solution provided for this purpose. If the lens appears to be damaged, do not use it.
- Rinse your lens. Place the lens in the palm of your hand and rinse it with the contact lens solution recommended by your specialist to remove any impurities. It is strictly forbidden to rinse your contact lenses with tap water.
- Put your Orthokeratology lens on your index finger and instill 1 or 2 drops of comfort solution recommended by your ophthalmologist or optometrist.
- Look at yourself in a mirror. Use your middle finger to spread the lower eyelid and your other hand to spread the upper eyelid.
- Approach the lens to your eye and gently place it on the cornea. Gently let go of the eyelids and blink several times to remove excess tears.
- Control your lens centering on your eye. If you notice air bubbles, a centering problem or discomfort, remove the lens and rest it using the same procedure.

### Lens removal

- Plunger use is recommended to remove your Orthokeratology lens
- Instill saline solution few drops in each eye and make sure the lens is mobile.
- Hold the plunger between your thumb and forefinger and look in the mirror. Position your middle finger just below the border of the lower eyelid and the index finger of the other hand on the border of the upper eyelid.
- Place the plunger in lens center and remove it.

### Lens care

- Every day rub your lens for about 20 seconds with a rigid lens cleaning solution and then rinse it with a saline solution.
- Before decontaminating your lens, empty your lens case. Never reuse the solution that was left in the case.
- Place your lens in its case and fill it with a hard lens soaking solution following this solution instructions recommendations.
- After not using lenses for a week, decontamination solution in the case must be renewed before placing the lenses on the eyes.
- Read and follow the instructions of the contact lens solutions used.
- Follow your specialist's recommendations.

### Precautions for use

- Orthokeratology rigid lenses are meant to be worn at night, while you sleep, eyes closed: put them in the evening within 30 minutes of bedtime and remove them in the morning within 15 minutes of getting up.
- Do not use your saliva or any household products to maintain your lenses: only use the rigid lens care solutions recommended by your specialist.
- Do not change the brand of your contact lens solution without consulting your specialist.
- Do not put your lenses in contact with any cosmetic aerosol product.
- Pregnancy and some medications can disrupt tear secretion: talk to your specialist.
- Do not wear your lenses if you are exposed to dangerous or irritating fumes
- Lenses wearing is not recommended in case of eye pathology
- Always carry eyeglasses (in case you must remove your lenses).
- Contraindications and undesirable side effects**
  - If you feel abnormal discomfort, remove your lens, and rinse it with a contact lens rinse solution.
  - Eye complications related to wearing lenses are rare if you follow the recommendations of professionals. However, if the following symptoms occur, remove your lenses, and contact your specialist:
    - A red or irritated eye,
    - A feeling of discomfort or persistent discomfort,
    - An abnormal change in your vision,
    - Increased sensitivity to light,
    - Watery eyes or dryness feeling.

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