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|  |
|--|
| <p><b>SOFT LENSES TRABECULENS INSTRUCTIONS FOR USE</b></p> |
|--|

This notice contains important information, read it, and keep it. TRABECULENS soft contact lenses are medical devices and should always be recommended and prescribed by your eye surgeon. For proper use, it is important that you follow all your specialist’s instructions and guidance on your lenses packaging and cleaning products.

- Your TRABECULENS soft lenses are permanently worn within the strict limit less than 1 month of wear and must be discarded after use.
- Your TRABECULENS soft lenses can only be prescribed by your eye surgeon.
- Wash your hands systematically before handling your lenses.
- TRABECULENS soft lenses are packaged in a sterile bottle containing a saline solution. The bottle is equipped with an inviolability system until the first opening. If the packaging is damaged, the lens should not be used.

### Intended use

TRABECULENS soft lenses are intended to be placed after glaucoma filter surgery in the event of:

- Very narrow anterior chamber of the eye

- Seidel post-operative

- Very low intraocular pressure.

Ophthalmologists are the only professional authorized to prescribe TRABECULENS soft lens wearing.

### Lens insertion

- Generally, it is your eye surgeon who puts the lens on your eye after the surgery. But you may have to place the lens yourself.
- TRABECULENS soft lenses are delivered sterile and full packaging must be checked before first use. Make sure the lens is clean and in good condition. If it looks damaged, do not use it.
- Rinse your lens. Place the lens in the palm of your hand and rinse it with a sterile saline solution. It is strictly forbidden to rinse your contact lenses with tap water.
- Look at yourself in a mirror. Use your middle finger to spread the lower eyelid and your other hand to spread the upper eyelid.
- Approach the lens to your eye and gently place it on the cornea. Gently let go of the eyelids and blink several times to remove excess tears.
- Control the centering of your lens on your eye. If you notice air bubbles, a centering problem or discomfort, remove the lens and rest it using the same procedure.

### Lens removal

- Look at yourself in a mirror.
- Spread your eyelids with your fingers
- Gently place your thumb and index finger on the lens that is on your eye. Pinch gently to remove the soft lens.

### Lens care

- The lens is intended to be permanently worn on the eye and to be discarded after a period of permanent wearing, and therefore should not need to be maintained. However, if your specialist tells you about a temporary lens removal, you can maintain your TRABECULENS soft lens like a traditional soft lens. So, if your specialist recommends a daily lens removal, follow the recommendations below:
- 20 seconds daily massage for your lens is recommended in the palm of your hand with a multifunctional solution for soft contact lenses, or, in case of greasy tears or deposits, with a soft lens cleaning product exclusively, then rinse it with a saline solution.
- Before decontaminating your lens, empty your lens case. Never reuse the solution that was left in the case.
- Place your lens in its case and fill it with multifunctional solution for the maintenance of soft lenses. Consider placing the right lens in the R (Right) and left compartment in the L (left) noted compartment.
- After a week without using lenses, decontamination solution in the case must be renewed before placing the lenses on the eyes.
- Read and follow maintenance instructions for solutions used.
- Follow your specialist’s recommendations.

### Precautions for use

- TRABECULENS soft lenses are intended to be worn permanently on the eye for a period-time defined by your eye surgeon (from a few days to 30 days maximum).
- Lens should never stay dry.
- Do not use your saliva or household products to maintain your lenses: only use the soft lens soaking solutions recommended by your specialist.
- Do not change your cleaning contact lens solution brand without consulting your specialist.
- Do not put your lenses in contact with any cosmetic aerosol product.
- You can keep your lenses to sleep if your specialist has indicated a permanent wear for several days.

### Contraindications and adverse effects

If you feel abnormal discomfort, remove your lens, and rinse it with a contact lens rinse solution.

Eye complications related to wearing lenses are rare if you follow the recommendations of professionals. However, if the following symptoms occur, remove your lenses, and contact your specialist :

- A red or irritated eye,

- A feeling of discomfort or persistent discomfort,

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